English

Deutsch

Français

Italiano

Nederlands

eCharger

User Guide • Gebrauchsanweisung • Guide
d'utilisation • Istruzioni d'uso • Gebruikershandleiding

SIEMENS

English	3
Deutsch	19
Français	35
Italiano	51
Nederlands	67

Content

Your Charger	4
Intended use	5
Important notes	6
Notes for batteries	9
How to charge	10
Useful tips for charging	14
Maintenance and care	15
Technical Data	16
Disposal information	18

EN

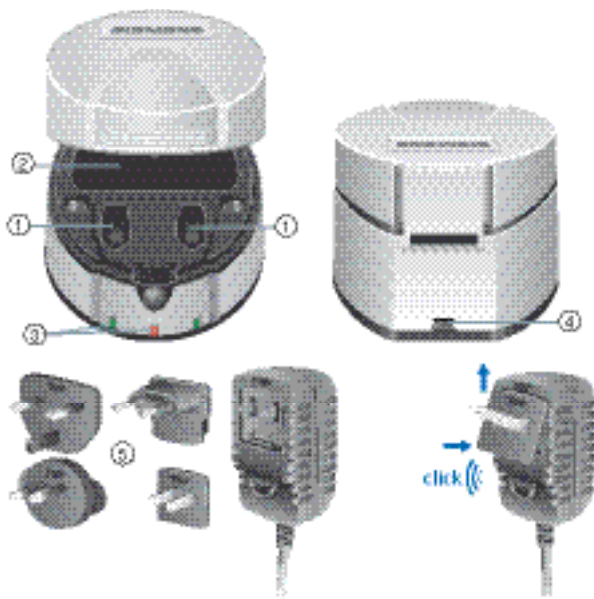
DE

FR

IT

NL

Your Charger

- ① Charging slots with charging contacts
- ② Cavity for receiver units, LifeTubes, domes and ear-molds
- ③ LEDs indicating the functional states of the charger
- ④ Contact for power supply
- ⑤ Power supply with country-specific adapters

Intended use

This charger is intended to charge Siemens hearing instruments.

Read and follow the instructions of this user guide to avoid injuries or damage to the charger and the hearing instrument.

Symbols:	
Be aware to information marked with the warning symbol WARNING, CAUTION or NOTICE!	
	WARNING points out a situation that could lead to serious injuries, CAUTION indicates a situation that could lead to minor and moderate injuries.
NOTICE	NOTICE indicates possible property damage.
	Advice and tips on how to handle your device better.
	Instruction. Indicates that something has to be done.

EN

DE

FR

IT

NL

Important notes

NOTICE

The charger is sensitive to extreme heat and high humidity.

- ▶ Do not expose the charger to extreme temperatures or high humidity.
- ▶ Do not expose to direct sunlight.

WARNING

Choking hazard posed by small parts.

- ▶ Keep hearing instruments, batteries and accessories out of children's reach.
- ▶ If swallowed consult a physician or hospital immediately.
- ▶ If infants, small children or mentally disabled persons need to wear hearing instruments ensure adequate supervision.

WARNING

Risk of injuries caused by electric shock!

- ▶ Do not touch the charging contacts in the charger.
- ▶ If cleaning is required, disconnect the charger from the power supply.

**CAUTION**

Risk of malfunction through incompatible parts.

- ▶ Use only the power-supply and country-specific adapter that is provided with the charger.

- This product does not have a power switch. It is turned on when the power cord is plugged in.
- The charger is intended only for indoor use.

**DANGER**

Incorrect use of the rechargeable battery can cause the rechargeable battery to burst. Discharging chemicals are spontaneously inflammable. Risk of overheating, explosion and fire!

- ▶ Do not ignore the red LED on the charger! (refer to the LED status table).
- ▶ Replace the rechargeable battery when its performance reduces considerably (running time under 8 hours).
- ▶ Do not use the rechargeable batteries when the expiration date on the packaging is exceeded.
- ▶ Do not use damaged or deformed rechargeable batteries.

EN

DE

FR

IT

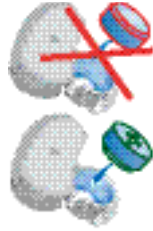
NL

- ▶ Do not open rechargeable batteries.
- ▶ Do not use rechargeable batteries in explosive environments.
- ▶ In the unlikely event if a burst rechargeable battery, keep all parts away from fire and everything that is readily combustible. Return the parts to your Hearing Care Professional for disposal.

Notes for batteries

- ▶ Use only rechargeable batteries that are provided by your Hearing Care Professional.
- ▶ Do not use the rechargeable batteries when the expiration date on the packaging is exceeded.
- ▶ For information about battery size check the user guide of your hearing instruments.
- ▶ Make sure the rechargeable batteries are inserted correctly.
- ▶ Charge the rechargeable battery before first use.

EN

DE

FR

IT

NL

NOTICE

Batteries contain harmful substances that pollute the environment.

- ▶ Do not throw used batteries into household trash.
- ▶ Dispose batteries according to national regulations or return them to your Hearing Care Professional.

How to charge

Preparation

- ▶ Connect the charger to a power supply.

Charging

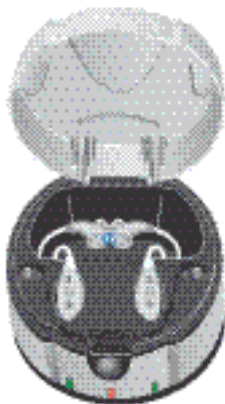
- ▶ Press the front button to open the charger.
The lid unlocks.
- ▶ Place your hearing instruments into the charging slots.
- ▶ Close the lid of the charger.

The hearing instruments are switched off automatically and the charging process starts.

The green LED is blinking.

Charging is complete when the green LED stays on continuously.

- ▶ Remove the hearing instruments from the charger when you next want to use them.
- ▶ Check if the hearing instruments are turned on.

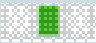
LED status

The charger's LEDs indicate the charging status.

Middle LED: Connection to the power supply

Left/right LED: Charging process of left / right hearing instrument

LED and state	Explanation
Middle LED shining green 	Charger is connected to power supply. Charger is ready to use.
Middle LED is off 	<ul style="list-style-type: none"> ▶ Check whether charger is connected to the correct power supply.
Left/ right LED blinking green 	Charging process of left/right hearing instrument is ongoing. <ul style="list-style-type: none"> ▶ Leave hearing instrument in charger.
Left/ right LED shining green 	Charging process of left/right hearing instrument is finished. Charger preserves the charge of the battery. <ul style="list-style-type: none"> ▶ Remove hearing instruments from charger when you want to use them.

EN

DE

FR

IT

NL

LED and state	Explanation
<p data-bbox="149 358 353 423">Left and right LED shining red</p> 	<p data-bbox="397 358 925 423">Defect inlay in the charger. Charging can not be initiated.</p> <ul data-bbox="397 438 925 620" style="list-style-type: none"><li data-bbox="397 438 925 620">▶ Insert the correct inlay for your hearing instruments / ask your Hearing Care Professional to insert the correct inlay for your hearing instruments.
<p data-bbox="149 646 342 711">Left / right LED shining red</p> 	<p data-bbox="397 646 876 711">Wrong polarity of battery in left/right hearing instrument.</p> <ul data-bbox="397 726 925 877" style="list-style-type: none"><li data-bbox="397 726 925 762">▶ Check polarity of batteries.<li data-bbox="397 770 925 877">▶ Check whether hearing instruments are inserted correctly in charging slots.
<p data-bbox="149 899 342 965">Left / right LED blinking red</p> 	<p data-bbox="397 899 925 1001">Wrong battery in left/right hearing instrument. Charging process can not be initiated.</p> <ul data-bbox="397 1016 925 1052" style="list-style-type: none"><li data-bbox="397 1016 925 1052">▶ Check type of batteries.

LED and state	Explanation
	<p>If type and polarity of batteries are okay but a red LED is apparent:</p> <ul style="list-style-type: none"> ▶ Ask your Hearing Care Professional to clean the contacts of the hearing instruments and charger. ▶ Ask your Hearing Care Professional to check the functioning of the rechargeable batteries and the charger.
<p>Left / right LED blinking orange</p> 	<p>External temperature too high. The charging process is interrupted to protect the functionality of the rechargeable batteries. charging continues when the external temperature drops under 45°C (113°F).</p> <ul style="list-style-type: none"> ▶ Take the charger to a cooler place.

EN

DE

FR

IT

NL

Consult your Hearing Care Professional if you encounter problems.

Useful tips for charging

A good maintenance of your rechargeable batteries will enhance their life time.

- ▶ Avoid charging above 30° C (86°F). It reduces the battery life time.

The rechargeable battery has no memory effect.

- ▶ Charge the battery on a regular basis (e.g. once a day) thus avoiding running it down completely.

A charging cycle takes about 6 hours

If you need to remove the hearing instruments before the charging cycle is complete, the operation time can be shorter than usual.

NOTICE

The rechargeable battery will be damaged or unusable when deep discharged.

- ▶ Only leave the hearing instruments in the charger when the charger is connected to a power supply.
- ▶ Remove the rechargeable batteries from the hearing instruments if you intend not to use them for several days. Charge the rechargeable battery before.

Maintenance and care

NOTICE

The charger will be damaged by dissolver.

▶ Do not sterilise or disinfect the charger.

- ▶ Use a dry, soft tissue to clean the charger.
- ▶ Ask your Hearing Care Professional if the charging contacts need cleaning.

Dry function

The charger has an electronic drying function reducing the moisture in your hearing instruments.

EN

DE

FR

IT

NL

Technical Data

Specifications Charger 3G	
Operating temperature:	15...35 °C / 59 ... 95°F
Operating humidity:	20% ... 75%
Minimum air pressure:	70 kPa
Storage temperatur:	-10 ... 40 °C / 14 ... 104 °F
Storage humidity:	20% ... 75%
Input voltage:	100 - 240 V AC
Input current:	< 0.5 A
Protection against electric shock: class II EMI: complies with EN 60601-1-2. EN 55011, class B Against ingress of water: IP XO (ordinary)	
Safety approvals: <div style="display: flex; align-items: center; justify-content: center;">    <div style="margin-left: 10px;"> <p>MEDICAL EQUIPMENT WITH RESPECT TO ELECTRICAL SHOCK, FIRE, AND MECHANICAL HAZARDS ONLY; IN ACCORDANCE WITH UL 60601-1 CAN/CSA C22.2 No. 601.1 3JFC</p> </div> </div>	

Specifications rechargeable batteries p312accu, p13accu; provided by Power One	
Operating temperature (in hearing instrument)	-20 ...60°C / -4 ... 140 °F
Storage temperature (average)	10 ... 26 °C / 50 ... 79 °F

Technical data - Country-specific power-supply

Globtek Model GT-41076-0605	
Input voltage:	100-240 V AC
Input current:	0.3 A RMS MAX
Input frequency:	50-60 Hz
Output voltage:	5 VDC
Output current:	1.2 A

Kuantech Model KSAA0500080W1UV-1	
Input voltage:	100-240 V AC
Input current:	0.18 A RMS MAX
Input frequency:	50-60 Hz
Output voltage:	5 VDC
Output current:	0.8 A

Fuhua Model: UE10W-050080SPC	
Input voltage:	100-240 V AC
Input current:	0.15 A RMS MAX
Input frequency:	50-60 Hz
Output voltage:	5 VDC
Output current:	0.8 A

EN

DE

FR

IT

NL

Disposal information

Within the European Union, the marked equipment is covered by "Directive 2002/96/EC of the European Parliament and of the Council of 27 January 2003 on waste electrical and electronic equipment."

Amended by "Directive 2003/108/EC" (WEEE).

▶ Recycle the device and accessories according to national regulations.

With the CE marking Siemens confirms compliance with the European Directive 93/42/EEC concerning medical devices.

With the CE marking Siemens confirms compliance with the European Directive 2004/108/EC concerning electromagnetic compatibility.

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- ▶ Do not ignore the red LED on the charger! (refer to the LED status table).
- ▶ Replace the rechargeable battery when its performance reduces considerably (running time under 8 hours).
- ▶ Do not use the rechargeable batteries when the expiration date on the packaging is exceeded.

- ▶ Do not use damaged or deformed rechargeable batteries.
- ▶ Do not open rechargeable batteries.
- ▶ Do not use rechargeable batteries in explosive environments.
- ▶ In the unlikely event if a burst rechargeable battery, keep all parts away from fire and everything that is readily combustible. Return the parts to your Hearing Care Professional for disposal.

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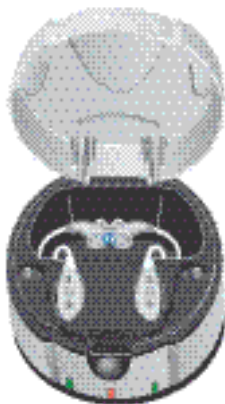
IT

NL

LED and state	Explanation
<p data-bbox="149 355 357 423">Left and right LED shining red</p> 	<p data-bbox="381 355 912 423">Defect inlay in the charger. Charging can not be initiated.</p> <ul style="list-style-type: none"> <li data-bbox="381 433 897 623">▶ Insert the correct inlay for your hearing instruments / ask your Hearing Care Professional to insert the correct inlay for your hearing instruments.
<p data-bbox="149 640 346 709">Left / right LED shining red</p> 	<p data-bbox="381 640 864 709">Wrong polarity of battery in left/right hearing instrument.</p> <ul style="list-style-type: none"> <li data-bbox="381 719 788 758">▶ Check polarity of batteries. <li data-bbox="381 768 897 875">▶ Check whether hearing instruments are inserted correctly in charging slots.
<p data-bbox="149 893 346 962">Left / right LED blinking red</p> 	<p data-bbox="381 893 912 1000">Wrong battery in left/right hearing instrument. Charging process can not be initiated.</p> <ul style="list-style-type: none"> <li data-bbox="381 1010 746 1049">▶ Check type of batteries.

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NL

Technical Data

Specifications Charger 3G	
Operating temperature:	15...35 °C / 59 ... 95°F
Operating humidity:	20% ... 75%
Minimum air pressure:	70 kPa
Storage temperatur:	-10 ... 40 °C / 14 ... 104 °F
Storage humidity:	20% ... 75%
Input voltage:	100 - 240 V AC
Input current:	< 0.5 A
Protection against electric shock: class II EMI: complies with EN 60601-1-2. EN 55011, class B Against ingress of water: IP XO (ordinary)	
Safety approvals: <div style="display: flex; align-items: center; justify-content: center; margin-top: 10px;">    <div style="margin-left: 20px; font-size: small;"> MEDICAL EQUIPMENT WITH RESPECT TO ELECTRICAL SHOCK, FIRE, AND MECHANICAL HAZARDS ONLY ; IN ACCORDANCE WITH UL 60601-1 CAN/CSA C22.2 No. 601.1.3 JFC </div> </div>	

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© Siemens AG, 02.2010 · SB
Siemens Audiologische Technik GmbH